
# UNWIREDTHINGS IKÖNN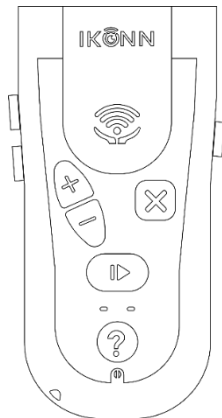

**EN** Instruction manual  
**DA** Instruktionsmanual  
**NO** Instruksjonsmanual  
**SV** Bruksanvisning  
**FI** Käyttöohjeet  
**IS** Leiðbeiningarhandbók

|                                      |   |
|--------------------------------------|---|
| Symbols/Symboler/Symbolit/Tákn ..... | 4 |
|--------------------------------------|---|

## EN

|                                               |    |
|-----------------------------------------------|----|
| Preface .....                                 | 5  |
| Device description and intended purpose ..... | 5  |
| Before first use .....                        | 5  |
| Parts and components.....                     | 6  |
| Operation .....                               | 6  |
| Mobile App.....                               | 13 |
| Tips and troubleshooting.....                 | 13 |
| Warnings, handling and safety .....           | 14 |
| Technical specifications .....                | 16 |
| Support and warranty .....                    | 17 |
| Disposal of electronic appliances .....       | 17 |
| Copyright.....                                | 17 |
| Declaration of conformity .....               | 18 |

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## Preface

---

### **Congratulations on purchasing your IKONN® Controller.**

This manual serves as a guide to efficient and safe use of the product. Read this information thoroughly before using the product and store this document for future reference.

## Device description and intended purpose

---

The IKONN Controller is designed for universal use, providing greater accessibility and independence for individuals with reduced vision. It features built-in functions such as a clock, timers, alarms, voice memos, and games. IKONN connects to a variety of compatible household devices to read back their measurements. In addition to wireless accessories, it also supports plug-in sensors that connect directly to the controller. Acting as both a remote control and an audio guide, it ensures everyone, including those unable to rely on a screen, can easily access essential information and controls.

## Before first use

---


Unpack the product with care. All packaging materials are recyclable, however you may wish to keep the packaging until you are satisfied that IKONN meets your needs.

Having taken the product out of the packaging, ensure that all the parts and components are present and in good condition. In case of any missing elements, please get in touch with the retailer from whom you purchased the product.

**Warning:** To avoid injury or damage to the product, read section *Handling and safety*, before operating the product for the first time.

## Parts and components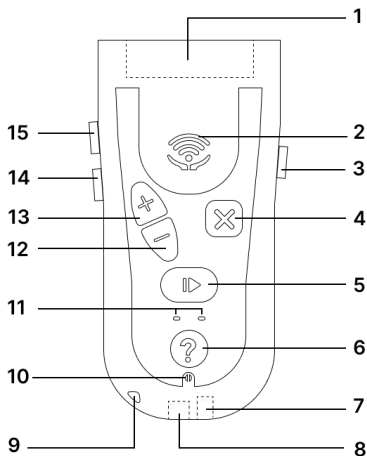

| Item | Symbol                                                                                                   |
|------|----------------------------------------------------------------------------------------------------------|
| 1    | Accessory port                                                                                           |
| 2    | Audio speaker                                                                                            |
| 3    | Power button [  ]       |
| 4    | Cancel button [  ]      |
| 5    | OK button [  ]          |
| 6    | Help button [  ]        |
| 7    | Audio jack port                                                                                          |
| 8    | Charging port                                                                                            |
| 9    | Lanyard holder                                                                                           |
| 10   | Microphone                                                                                               |
| 11   | LEDs                                                                                                     |
| 12   | Minus button [  ]       |
| 13   | Plus button [  ]        |
| 14   | Volume down button [  ] |
| 15   | Volume up button [  ]   |

## Operation

### Get started

To turn on the Controller, locate the single button on the right side of the device and hold it until you hear a beep (after about 3 seconds). To turn the device off, hold down the same button. Your device may need to be charged before use, in which case you should use the supplied USB-C cable, referring to the section on *Charging*.

After powering up, IKONN is in idle mode, with the Clock function selected. If you press the OK button briefly, you will hear the time. If you hold down the Action button for a longer time and release it, you will hear the date instead. In this way, pressing the buttons for a shorter or longer time can change their functionality.

| Expression  | Meaning                                        |
|-------------|------------------------------------------------|
| Short press | Pressing the button for half a second          |
| Long press  | Pressing the button for about 1 second or more |
| Hold down   | Pressing the button for more than 2 seconds    |

The functions available on IKONN are presented in an audio menu. You can scroll through the list using the Plus [ **+** ] and Minus [ **-** ] arrow buttons. Once you have found the function you want to use, press the OK button [ **▶** ] to select it which you also use to confirm choices. Pressing the square Cancel [ **✕** ] takes you back a step in a menu or stops the current activity.

If you want more detailed information about the function, you can long-press the Help button [ **?** ] to receive an explanation of how to use the function or accessory. Some options will only be available if you have connected a specific physical accessory such as a color sensor or purchased and added a product such as a kitchen scale.

All functions behave in a similar way. At any point you can leave a function by holding down the square Cancel [ **✕** ] button for a second to return to the main menu. If you're unsure what to do at any point, you can press the circular Help [ **?** ] button to hear the information guide.

To preserve battery life, the IKONN Controller will automatically enter sleep mode after a period of inactivity—provided it is not charging or connected to another device. You can wake it up by pressing any key. If you place it in a pocket or bag where the buttons may be bumped,

consider using the key lock function. If you know the device won't be used for an extended period, consider turning it off completely..

| In order to                                                                                      | Activity                                                                                                                                                                                                                                                                                                |
|--------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Turn the device on or off                                                                        | Hold down [  ]                                                                                                                                                                                                         |
| Scroll through the list of available apps or increase/decrease a value when setting e.g. a timer | Use [  ] and [  ]                                                                                                                     |
| Adjust volume                                                                                    | Use [  + ] and [  - ]                                                                                                                 |
| Access chosen app. Start or stop a function in an app                                            | Short press [  ]                                                                                                                                                                                                       |
| Launch an alternative action for an app                                                          | Long press [  ]                                                                                                                                                                                                        |
| Return to a previous step or cancel current activity                                             | Short press [  ]                                                                                                                                                                                                       |
| Cancel all activity and return to default state                                                  | Long press [  ]                                                                                                                                                                                                        |
| Get guidance on a particular function or accessory                                               | Press [  ]                                                                                                                                                                                                             |
| Dismiss alerts such as timer or alarm expiry                                                     | Short press [  ] or [  ]                                                                                                              |
| Lock and unlock the keypad                                                                       | Hold down [  ] and then click [  ]. Release [  ] key |

## Built-in functionality

The default IKONN features are available immediately after turning the device on. Amongst these is a **training program** which helps to familiarize you with operating the product.

It is recommended that you listen to the training program to get an overview of the controls and functions. After turning on the device for the first time, you can select it by pressing the down arrow key once. When you hear “*Training*”, short press the OK button [  ]. Follow the voice guide, which will help you get more acquainted with the device. The training course takes about 12 minutes to complete, but you can take as much time as you want and revisit the content by stepping backward.

In addition to the training program, you can access the following functions from the main menu:

**Clock** – Announces the current time in **12-hour, 24-hour, or analog format**, along with the date.

**Alarm Clock** – Allows you to set up to **five alarms**, either as a one-time alert or with recurring options for a selected weekday, or every day.

**Voice Memo** – Enables you to **record and store short voice messages**, which can serve as personal reminders.

**Timer** – Lets you set up to **three individual timers**, which can be stopped and restarted as needed.


**Add New Product** – Connects your IKONN Controller to compatible devices. For more details, refer to the sections *Compatible Products* and *Adding a New Product* of the manual.

**Games** - Offers simple, interactive features such as a virtual **coin toss** and **dice roll**, helping you to make critical life decisions or participate in board games.

**Settings** – Allows you to set and customize various aspects of your IKONN Controller. This includes:

- Setting the date and time and choosing your preferred **time format**.
- Changing the **units** that IKONN gives measurements in.
- Checking the device's **battery level**.
- Adjusting how much IKONN automatically provides help in each function.
- Specifying your **height** for BMI measurement.
- Resetting your device to **remove** all installed products, settings and voice memos.


## Charging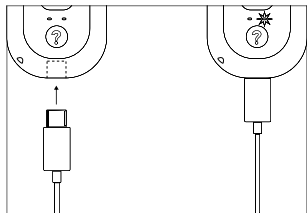

To charge the IKONN Controller, plug the thin USB-C end of the USB-C cable into the charging port at the base of the device. The other rectangular end (USB-A) can be plugged into a charger or a computer. IKONN will let you know when its battery is getting low and needs charging. Charging progression is indicated by an orange LED light on the keypad. When IKONN is fully charged it will inform you verbally. When charging has completed, we

recommend removing IKONN from the charging cable. Even if charging has not completed, you can always remove the cable if you need to use your device.

## Plug-in accessories

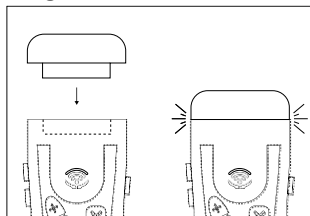At the top of the device, you'll find the wide curved module **docking port** where you can attach plug-in accessories. When you first take the device out of the box, you'll find a silicone plug protecting the docking port - be sure to remove this piece before connecting any accessories. When you attach an accessory, new functions may become available in the menu, such as a color sensor or light detector. When you remove an

accessory, the associated functions will no longer be available. Take an accessory and notice that the base has some protruding rails. These rails slide into the grooves on the lower surface of the docking port. Push the accessory in gently until you feel a click. When you've connected the accessory, you'll hear a sound and message indicating which module has been attached. When you remove the accessory, you will also hear a message. Reinsert the protective silicone plug to protect the docking port.

**Warning:** Do not insert anything else into the accessory docking port, as it may cause damage to the port or to the device. Never force the accessory; if it doesn't slide in with gentle pressure, it is probably upside down.

## Microphone

The IKONN Controller contains a microphone at the base of the unit, just below the help button. The microphone is used when recording voice memos or to name items such as timers or alarms. Any functions which make use of the microphone require you to long press the OK [ ► ] button to start recording. During recording, short press OK to stop and save the recording or cancel to discard it.

## Compatible products

Besides connecting to plug-in accessories, IKONN also connects with a wide range of other products labeled as **IKONN compatible**. This expanding range of devices includes kitchen scales, bathroom scales, room thermometers, kitchen thermometers, tape measures and more.

To utilize one of these functions, select it from the menu using the Plus [ + ] and Minus [ — ] buttons and then press the OK button [ ► ]. Make sure the IKONN compatible product is turned on, and IKONN should immediately detect it. If you are uncertain about the functions available for the certain product, you can always long press the Help button [ ? ].

## Adding a new product

To avoid presenting unnecessary options, IKONN only lists products you have installed. To install a new product, choose the “*Add new product*” entry in the main menu. Press OK [ ► ] to scan for it, and ensure that the new product is turned on and nearby. Only turn on one new product of the same type at a time. IKONN will locate the product and ask if you wish to install the product. Press the Action button to confirm or Cancel if the wrong product was

EN

found and you need to re-scan for the new product. If you cannot find your new product, ensure that it has batteries and is powered on.

Note that some products may come with batteries pre-installed, in which case you may need to pull out the small plastic insulating tab before operation.

You only need to perform this installation action once with a new product. After doing so, the IKONN Controller will restart automatically, and the product will be added to the list of options in the main menu between the Clock and Training entries. When in Clock mode, press the down arrow key to select an accessory. You can also install the same product to several IKONN controllers if you share it with others and you can install multiple products of the same type to the controller in which case IKONN will select whichever device it finds first when searching.

## Headset connection

The 3.5mm audio jack for headphones is located on the bottom edge of the device, near the USB-C charging connector. Plugging in headphones will redirect audio from the speaker to the headphones, so if no sound is heard from the speaker, check if headphones are connected. This version of IKONN only supports 3.5 mm audio jacks, and earphones with different plugs, such as those from early iPhone models, are not compatible.

Avoid listening at unnecessarily high volumes. Note that the Controller cannot drive headphones with very high impedances.

## Lanyard

**Warning:** The IKONN controller has a small hole in the enclosure allowing the mounting of a neck or wrist lanyard. Should you choose to mount your own lanyard ensure that it has a safety break-out function to avoid serious injury if the lanyard is suddenly pulled in an accident.

## Mobile App

---

Although IKONN is designed to function as a standalone device out of the box, it is recommended that you use it together with the dedicated *IKONN Update* smartphone app to maximize its capabilities.

This smartphone app connects with IKONN and provides access to system updates, the ability to change voices as well as **additional features** and **personalization options** to suit your preferences. The app also synchronizes the IKONN clock time to that of the phone network, allowing for very precise time setting.

### Download

Download the **IKONN Update** app, available on the App Store® and follow the on-screen instructions to connect with IKONN.

### Setup

Open the app and follow the instructions for connecting with your IKONN device which should be located close to the phone.

**Note that IKONN must be connected with a cable to a charger in order to establish a connection with the phone app.**

When connection is requested, the left blue LED on the IKONN will start **blinking** and you will be prompted by both the IKONN device and the app to approve the connection. The light will turn solid once you approve the request using IKONN.

### Tips and troubleshooting

---

You can find a list of solutions for the most frequently occurring issues at:  
[ikonn.info/products/ikonn-controller](http://ikonn.info/products/ikonn-controller)

If you experience problems in operating your product, please contact the retailer where you purchased it. If you are unable to make contact with them, contact the manufacturer at: [ikonn.info/support](mailto:ikonn.info/support)

## Warnings, handling and safety

---

**Warning:** Read all safety information below before using IKONN and keep this manual for future reference. As with all electronic products, failure to follow safety instructions can result in fire, electric shock, or other injuries, as well as damage to IKONN or other property.

### General safety and handling

This device should be used in compliance with the intended use. Handle IKONN with care, as it contains sensitive electronic components. IKONN can be damaged if dropped, burned, punctured, or crushed, or immersed in liquid. Always check for visible signs of damage. If you find any, reach out to the retailer from whom you purchased the product. Do not open or attempt to repair IKONN yourself, as disassembling of the device may cause injury to you or damage the device. Do not use a damaged IKONN device.

Charge IKONN with the included cable and optional adapter or other third-party made for IKONN cable and compatible power adapters. Using damaged cables or chargers or charging in the presence of moisture can cause an electrical faults or shocks. Ensure that the AC plug or AC power cord is fully inserted into a power outlet.

High volume sound levels may cause hearing damage. Do not listen to sounds at high volume levels for long periods. Check the volume before inserting anything in your ear.

Radio frequency emissions from IKONN can negatively affect the operation of other electronic equipment.

IKONN does not transmit any radio frequencies when in its base state, operating as a clock or alarm clock, however it uses radio functionality to advertise its presence when a charging cable is inserted. Do not charge IKONN or use it with compatible products in sensitive areas, such as an aircraft in flight.

Avoid applying excessive pressure or bending the device. Never force a connector into a port or apply excessive pressure to buttons. If the connector and port do not join with reasonable ease, check for obstructions, make sure that the connector matches the port and that the connector is positioned correctly in relation to the port.

Do not operate the device in places that are hot, humid, excessively dusty, where insects may enter, where it might be exposed to mechanical vibration, near flammable objects and in rooms with smoke or steam. Avoid leaving it in sunlight in a hot car for example.

Operate IKONN only in acceptable temperatures. IKONN's work and storage temperature range is given in the *Technical specifications* section. Operating it outside of this range may result in its damage and shortening of battery life. Avoid exposing IKONN to dramatic changes in temperature or humidity. It is normal for IKONN to feel slightly warm while charging or during prolonged use.

Ensure that IKONN remains dry at all times. Do not use the device when your hands are wet or expose it to water. In order to clean IKONN, unplug all cables, turn it off and use a soft, slightly damp, lint-free cloth. Avoid the use of alcohol or harsh abrasive cleaning agents. Avoid getting moisture in openings.

If you have medical conditions that you believe could be affected by IKONN, consult with your physician prior to using IKONN.

## Batteries

Do not attempt to replace the battery in IKONN yourself. The IKONN battery should be replaced only by an authorized service provider and must be recycled or disposed of separately from household waste.

EN

Do not burn or expose the batteries to high temperatures, as this could lead to explosions. Do not tamper with batteries or cells in any way.

If your IKONN product is destroyed by accident, place it in a sealed plastic bag prior to transporting it anywhere.

## Security

The device does not store personal identifiers. It uses strong hardware and IT security measures, including encryption and authentication, to protect against unauthorized access and data interception. Connections are limited to trusted networks, ensuring secure and reliable operation.

## Technical specifications

|                              |                                     |                                     |                             |
|------------------------------|-------------------------------------|-------------------------------------|-----------------------------|
| <b>Model</b>                 | IKONN-1                             | <b>Dimensions</b>                   | 120x65x15 mm                |
| <b>Weight</b>                | 83g                                 | <b>RF TX Power</b>                  | Max 12dBm                   |
| <b>Power supply</b>          | Lithium-ion battery (non-removable) | <b>Transmission frequency range</b> | RF: ISM, 2.400 - 2.4835 GHz |
| <b>Battery lifetime</b>      | 300 charging cycles or 24 months    | <b>Charging voltage/current</b>     | 4.9V-5.1V, 500mA            |
| <b>Transport temperature</b> | from 10°C to 40°C                   | <b>Working temperature</b>          | from 5°C to 40°C            |
| <b>Transport humidity</b>    | from 0% to 85%                      | <b>Working humidity</b>             | from 0% to 85%              |
| <b>Modulation</b>            | DSSS                                | <b>Expected service life</b>        | 3 years                     |

Technical specifications may change without prior notification to improve product quality.

Made in EU.

## Support and warranty

---

**This product is warranted against defects in materials and workmanship for a period of two years from the date of original purchase.** This warranty does not cover damage caused by misuse, accident or unauthorized modifications. For parts subject to wear and tear, including the battery, the warranty is six months. Please retain your proof of purchase.

## Disposal of electronic appliances

---

The product and/or batteries should be disposed of separately from household waste. Upon reaching the end of its operational life, ensure the proper disposal of cells, batteries, or accumulators by delivering them to a collection point designated by your local authorities. Consumers are encouraged to reach out to their local authorities or the retailer for guidance on the correct disposal of old machines and their associated batteries. In the EU these rules are governed by directive 2012/19/EU (Waste Electrical and Electronic Equipment).

## Copyright

---

Unwired Things ApS retains intellectual property rights to the content contained within this document and all hardware design, audio content and software functionality contained in the IKONN product. All rights reserved. Unauthorized reproduction, distribution, or use of any part of this document and product may violate applicable laws and could result in civil and criminal penalties. IKONN is a registered trademark of Unwired Things. All rights reserved.

©Unwired Things 2025

## Declaration of conformity

---

The manufacturer, Unwired Things, declares that the IKONN-1 device complies with Radio Equipment Directive 2014/53/EU. The full text of the EU Declaration of Conformity is available online: [ikonn.info/conformity](http://ikonn.info/conformity)

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